

ENZY-CLEAN

Enzyme - Cleaner Concentrate

Application procedure for manual cleaning:

After using the endoscope, immediately draw up a hand warm aqueous Enzy Clean solution into the channels and subsequently immerse the whole instrument in the solution.

Please follow your endoscope producers' cleaning instruction as well as the cleaning and disinfection guidelines for GI-Endoscopy of the European Society of Gastrointestinal Endoscopy (E.S.G.E.).

Concentration of use:

Dependent on the bio burden: dissolve 1 to 5 % of Concentrate in handwarm tap water

Incubation period:

Dependent on the bio burden: 15 to 60 minutes

Further treatment:

After the enzyme treatment, rinse the endoscope carefully with tap water and disinfect it according to advised protocol.

ENZY-CLEAN is compatible with all disinfectants.

Important advice:

After disinfection the remaining no water may remain in the endoscope.

It is advisable to disinfect your endoscope prior to use if it has not been used for some time (more than 2 days), since remaining water can become contaminated within very short time!

Warning advice:

EnzyClean is keratolytic. Please prevent skin contact (= wear gloves)

Splashes on skin or eyes have to be rinsed off immediately and carefully.

Storage:

Store ENZY-CLEAN in cool condition. Application solution has to be rinsed of after 8 hours.

Environmental compatibility:

ENZY-CLEAN is easily biodegradable.

Producer:

CosmArt Laboratorium GmbH

Neuwiesenfeld 21

D- 64625 Bensheim

www.cosmart.de